MANUAL DE USO

● FORMA DE USO

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⚠ IMPORTANTE:

THE SPACER IS A MEDICAL DEVICE, SO MAINTAIN OUTSIDE THE REACH OF CHILDREN. FOLLOW THE INSTRUCTIONS OF YOUR DOCTOR.

● INSTRUCTIONS FOR USE

1. Open de cover of the pressurised aerosol inhaler and shake it during 10 seconds.
2. Insert the inhaler in the back of the spacer.
3. **a)** With FACE MASK: (For infants less than 6 years and grater people with coordination difficulties): Apply the face mask covering the mouth and noose.
3. **b)** With MOUTHPIECE: (For children greater than 6 years and adults): Put your mouth around the mouthpiece ensuring there are not gaps and press your lips.
4. **a)** With FACE MASK: Press the aerosol inhaler and then mantein the mask placed firmly on the face during 30 seconds, making sure that the patiens breathe regularly.
4. **b)** With MOUTHPIECE: Press the aerosol inhaler and then make two deep inhalations complete and slow. At the end, contain the breathing during 30 seconds and then exhale slowly.
5. Retain the breathing as much as it is possible (if you have difficulty to retain the breathing, exhale normally within the device 3 or 4 times and retake all the procedure again).
6. Repit steps 2 to 5 accordlly to the medical prescription of the number of dose that is due to administer. In case you need more than 1 dose of medication, start the second one, more than 60 seconds after the first.
7. When finished, remove the inhaler from the spacer and replace the protective cap.

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Importado y Distribuido por: Laboratorios CMC-Pharma SpA.